

MAD Ultrasonics

providing the finest ultrasonic inserts and service

Madalyn Rosenbluth, RDH
Founder



151 Winchester Drive, Yonkers, NY 10710 • (914) 844-6313
madultrasonics@yahoo.com • www.madultrasonics.com

GUIDELINES FOR OPTIMUM USAGE & PATIENT COMFORT

ALWAYS have unit on LOWEST POWER - turn up as needed

Manual units-keep power on low and tune frequency to vibration

Auto units-turn power to lowest and increase as needed for debridement

PLACE INSERT INTO HANDPIECE BEFORE activating foot pedal.

USE FAST WATER DRIP FLOW-for optimum, debridement, cavitation therapy and **PATIENT COMFORT/WATER COOLS TIP & TOOTH SURFACE**

ACTIVATE INSERT BEFORE APPLYING TO TOOTH SURFACE-NEVER WHILE ON IT

WORK WITH QUICK BRUSH STROKE MOTION-ALWAYS HAVE TIP MOVING

NO HARD PRESSURE AGAINST TOOTH WITH TIP-LIMITS ELIPTICAL MOVEMENT OF TIP

INSERT IS MOST ACTIVE AT TIP - AVOID DIRECT CONTACT OF TIP TO THE TOOTH, AS IT CREATES DAMAGE TO SURFACE

USE ALL SIDES OF TIP-LEAST ACTIVE TO MOST; BACK, LATERAL SIDES, WITH CONCAVE SIDE MOST ACTIVE